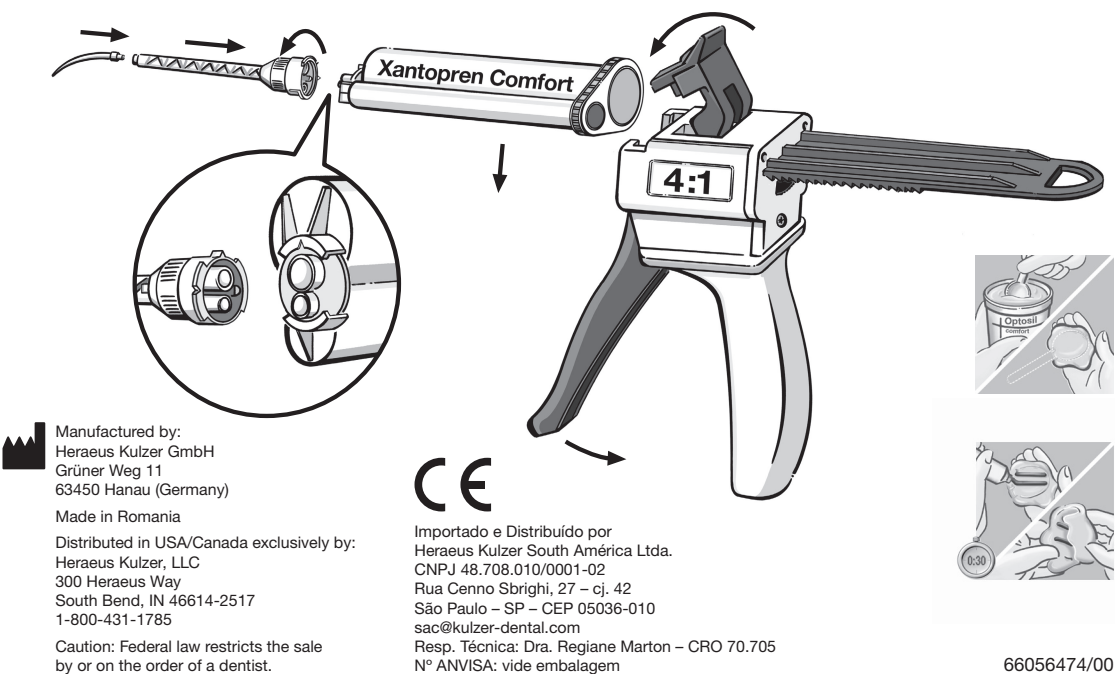
Manufactured by:
Heraeus Kulzer GmbH
Grüner Weg 11
63450 Hanau (Germany)
Made in Romania
Distributed in USA/Canada exclusively by:
Heraeus Kulzer, LLC
300 Hereaus Way
South Bend, IN 46614-2517
1-800-431-1785
Caution: Federal law restricts the sale
by or on the order of a dentist.

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Technical Information

Silicon based precision condensation curing impression material.

Optosil®/ Xantopren® Comfort®	EN ISO 4823	Colours	Dosage			Mixing- time	Working- time**	Time in mouth
			BASE	Activator universal Plus				
				Liquid	Paste			
			s	min : s	min : s			
Optosil® Comfort® Putty	Type 0	yellow	1 spoon (10.7 g/9.5 ml)	6-8 drops (0.14 g/0.13 ml)	6-8 graduations** (0.32 g/0.29 ml)	30	1:00	4:00
Xantopren® Comfort® Medium	Type 3	green	///	///	///	///	2:30	3:30
Xantopren® Comfort® Light	Type 3	blue	///	///	///	///	2:00	3:00

*1 At room temperature of 23°C (73°F), 50 % rel. humidity.

Higher temperatures shorten, lower temperatures prolong these times.

*2 Graduation marks as indicated on the mixing pad and mixing spatula (1 graduation = 1.3 cm)

The stated dosage should be observed.

User Advice

The Optosil®/Xantopren® assortment is a complete system.

Prior to first use extrude some material from the cartridge. After use replace mixing tip by protecting cap (hygienics).

For best results usage with other silicon materials and Activators is not recommended.

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Stand: 2013-07

Pouring the model:

Pouring can take place 30 minutes after the impression is removed from the mouth. Impressions remain dimensional stable up to 7 days. Visually inspect for damage before first use. Damaged products must not be used.

Recommended model material:

All commercially available modeling stones can be used.
Our suggestion:
Class III Gips, MOLDANO®/MOLDADUR®
Class IV Gips, MOLDASTONE®/MOLDASYNT®

Electroplating:

Impressions may be copper or silver plated in the usual way.

Disinfection:

Impressions should generally be treated by immersion for 10 minutes* in 5,25 % Sodium Hypochlorite solution. Other silicon specific disinfectant solutions should be used as directed by the products manufacturer.

* Gelson Luis Adabo, DDS, PhD, Elaine Zamarotti, Renata Garcia Fonseca, and Carlos Alberto dos Santos Cruz, DDS, PhD
Effect of disinfectant agents on dimensional stability of elastomeric impression materials. J Prosthet Dent, 81, 621-4, 1999

Adhesion:

We always recommend the use of our Universal Adhesive.

Storage conditions:

Do not store above 25°C (77°F).

Warning Activator:

Flammable. Inhalation can cause health problems. Eyes, respiratory organs and skin can be irritated. In an emergency, rinse off immediately with copious amounts of water and consult an eye specialist. Avoid skin contact. Wear protective clothing. Wear suitable gloves (EN 374, category III, e.g. Dermatril from KCL, Germany). Do not swallow and ingest. If health problems arise after swallowing impression material, seek medical attention immediately. Intestinal blockage may arise in rare cases. Hypersensitive persons may experience an allergic reaction to the product. Not recommended for breast feeding patients and children under 6 years. For use by dental personnel only.

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